BOWMAN Dispensers

## **FM300–0212** Face Mask Dispenser – Universal – Cone



### **Companion Products (each sold separately):**

- FM100-0212: Face Mask Dispenser Universal Earloop
- FM200-0212: Face Mask Dispenser Universal Shield (shown below)



#### **Product Description:**

- Face Mask Dispenser Universal Cone
- Holds a variety of boxed cone style face masks
- Keyholes and screw holes for wall mounting
- Quartz Beige ABS Plastic

#### Primary Area for Product Usage:

• Anywhere face masks are needed

#### **Dispenses**:

- Box(es) no larger than 6.75"W x 8.5"H x 5.50"D (17.2 cm x 21.6 cm x 14.0 cm)
- Box(es) no smaller than 4.28"W x 7.00"H x 4.13"D (10.9 cm x 17.8 cm x 10.5 cm)


#### Package Specifications:

- Package Quantity: 6 per case
- 16"L x 12"W x 12"H (40.6 cm x 30.5 cm x 30.5 cm)
- 7.0 lbs (3.2 kg) approximated

**Product Specifications** (overall external dimensions):

- 6.75"W x 7.13"H x 5.75"D (17.1 cm x 18.1 cm x 14.6 cm)
- 0.7 lbs (0.3 kg) approximated

#### Mounting Specifications:

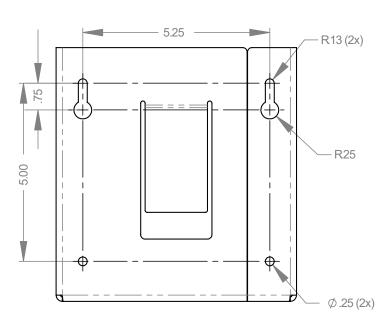

BOWMAN Dispensers


# FM300-0212 Face Mask Dispenser – Universal – Cone

Installation Instructions/Characteristics & Care

### Installation

Always mount dispensers securely prior to loading with product.

• Spring design provides a secure fit for a wide assortment of boxed products. • U-shaped front design provides a wide viewing space for face mask box

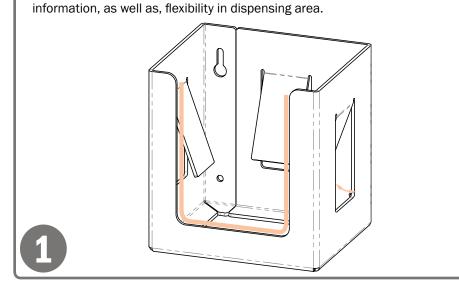


· Mount securely to a wall with screws approved and supplied by building maintenance.

**Characteristics & Care ABS** Plastic

Designed for Long-Term, Heavy-Duty Applications

The combination of copolymers (Acrylonitrile, Butadiene, and Styrene) gives ABS an excellent combination of strength, rigidity, and toughness.

ABS is chemical, stress-crack resistant, dimensionally stable, and will hold up to long-term use. It possesses high strength and performance characteristics at a reasonable cost.

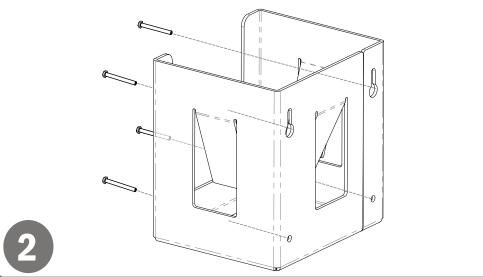

#### Parts included:

- Dispenser
- **Tools Required:** 
  - No tools required to assemble dispenser

Use mild soap and water to clean. Petroleum-based oils, solvents, and paints should not be used.

Standard hospital cleaning supplies are approved for use on our ABS plastic products.

Information subject to change without notice. Dimensions are approximate. (Not to scale)