

Patient Instructions for IBI Saliva Collection Kit



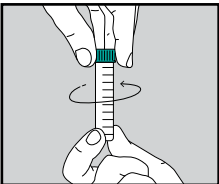
! DO NOT EAT, DRINK, SMOKE, BRUSH YOUR TEETH OR CHEW GUM AT LEAST 30 MINUTES BEFORE USE

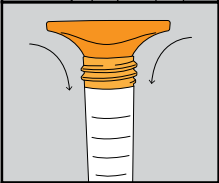
Kit Contents:

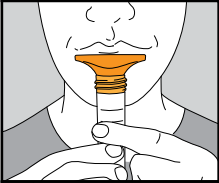
- 12 ml Collection tube
- Collection funnel

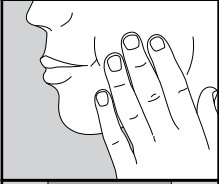
Before Collecting Saliva:

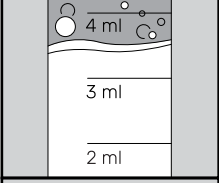
Remove the collection funnel and tube from the package

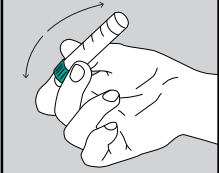
- 

1) Unscrew the cap from the collection tube. Keep the cap as it will be used to seal the sample.
- 

2) Push the funnel into the top of the tube. The funnel should fit snugly and bottom out on the top of the tube.
- 

3) Begin spitting into the funnel being careful not to spit too much at one time into the center of the funnel. Allow the spit to run down the tapered inside of the funnel so as not to vapor lock the tube.
- 

4) To promote saliva production rub the inside of your cheek with your tongue or gently massage the outside of your cheek.
- 

5) Continue to spitting into the funnel until the level in the tube reaches the fill line between 3 ml and 4 ml not including bubbles.
- 

6) Remove funnel and discard. Screw cap back on collection tube.