



**MYCO**  
**MEDICAL**

**TheSharpChoice<sup>®</sup>**

NMSDC Certified Diversity Supplier

DATASHEET

# QLICKSMART<sup>®</sup> BladeSINGLE Blade Remover

The Qlicksmart<sup>®</sup> BladeSINGLE is a single-handed, convenient, and safe blade removal system, providing a safe way to remove blades while continuing to use a standard handle.

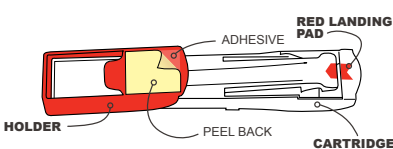
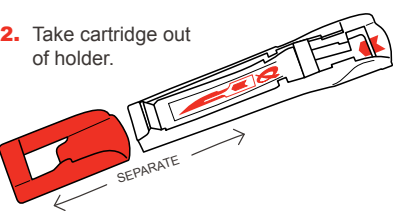
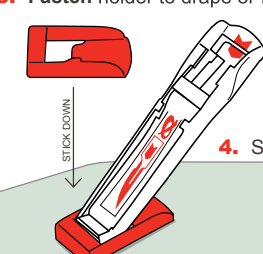

- OSHA compliant with single-handed activation
- Audible click to ensure proper placement
- Red arrows show proper insertion location
- Packaged sterile or non-sterile for flexibility in OR and ER settings
- Compatible with most brands of general surgery blades and flat handles



MYCO CODE	DESCRIPTION	UNIT OF MEASURE
QSBS-01	BladeSINGLE — Sterile	300 ea/bx; 4bx/cs
QSBS-01B	BladeSINGLE — Non-Sterile	2000 ea/case

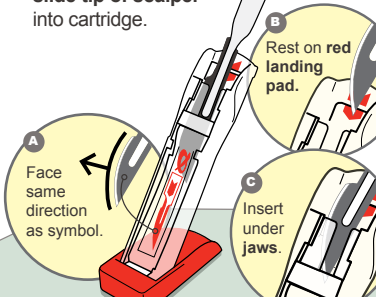
## How to Use:

### PREPARE USING TWO HANDS:

1. Peel to reveal adhesive.
 
2. Take cartridge out of holder.
 
3. Fasten holder to drape or flat surface.
 
4. Stand in holder.
 

**READY FOR USE**

### ACTIVATE SINGLE HANDED:

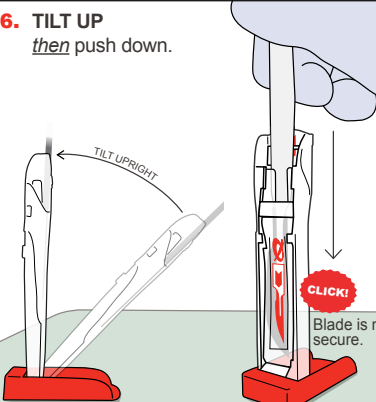
5. Using one hand, slide tip of scalpel into cartridge.
 

**SINGLE HANDED ACTION**

**A** Face same direction as symbol.

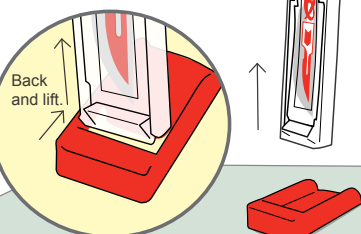
**B** Rest on red landing pad.

**C** Insert under jaws.

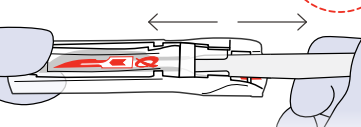
6. **TILT UP** then push down.
 


**TILT UPRIGHT**

**CLICK!** Blade is now secure.

7. Remove cartridge from holder.
 

### SAFE TO USE TWO HANDS:

8. Pull out scalpel handle.
 

**BLADE STAYS INSIDE CARTRIDGE**
9. Ready for counting and later disposal in sharps bin.
 

As a leading NMSDC Diversity Certified Supplier, MYCO Medical is committed to “Bringing Reliability, Innovation and Value” to the Healthcare industry with an emphasis on “physician preference” surgical devices and disposables.

To order, learn more or receive complimentary evaluation samples:

Please call MYCO Medical at (800) 454-6926 or (919) 460-2535 or email us at [info@mycomedical.com](mailto:info@mycomedical.com)



2015 Production Drive  
Apex, NC 27539 USA

[info@mycomedical.com](mailto:info@mycomedical.com)  
[www.mycomedical.com](http://www.mycomedical.com)

See the complete line of MYCO innovative disposable and reusable medical devices at [www.mycomedical.com](http://www.mycomedical.com)

Some medical devices are not licensed for sale in Canada.

© 2017 MYCO Medical, Inc. All rights reserved.  
LIT-00219 Rev 1 Eif 02.10.2017