Organize · Contain · Comply

# RE301-0012

# Semi-Recessed Glove Box Dispenser - Triple





## **Mounting Specifications:**

MADE USA

PATENT PENDING

Installation should be performed only by a licensed contractor.

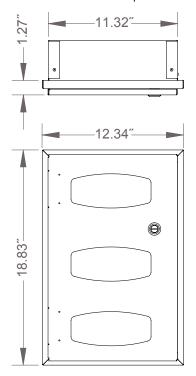

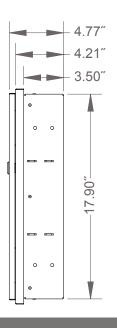

# **Product Description:**

- Semi-Recessed Glove Box Dispenser Triple
- · Holds 3 boxes of gloves
- · Low profile
- · Keyless locking mechanism
- · Screw holes in sides to mount to stud framing in wall
- Made of Quartz Beige Powder-Coated Steel

### Primary Area for Product Usage:

- All primary care areas: clinic/exam rooms, radiology, labs, physical therapy, O.R. areas, environmental services
- Any area where glove boxes need to be kept, but cannot protrude from the wall

### Dispenses:

- · Most manufacturers' standard glove boxes:
- Glove boxes no larger than 11.00"L x 5.50"W x 4.00"H (27.9 cm x 14.0 cm x 10.2 cm)

#### Package Specifications:

- · Package Quantity: 1 per case
- 24"L x 16"W x 6"H (61.0 cm x 40.6 cm x 15.2 cm)
- 11.0 lbs (5.0 kg) approximated

### **Product Specifications (overall external dimensions):**

- 12.34"W x 18.83"H x 4.77"D (31.3 cm x 47.8 cm x 12.1 cm)
- 9.4 lbs (4.2 kg) approximated

### **Alternative Products (each sold separately):**

- RE302-0012, Semi-Recessed Face Mask & Tissue Dispenser
- RE404-0012, Semi-Recessed Glove Box Dispenser Quad




# RE301-0012


# Semi-Recessed Glove Box Dispenser - Triple

Installation Instructions/Characteristics & Care

# Installation

Always mount dispensers securely prior to loading with product.

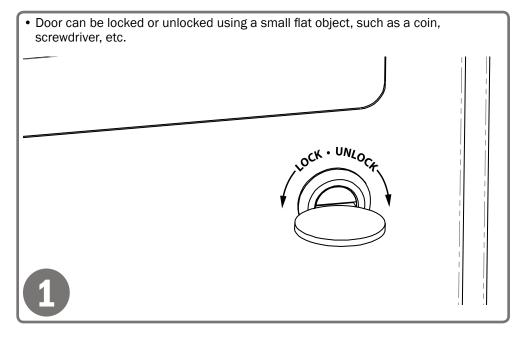

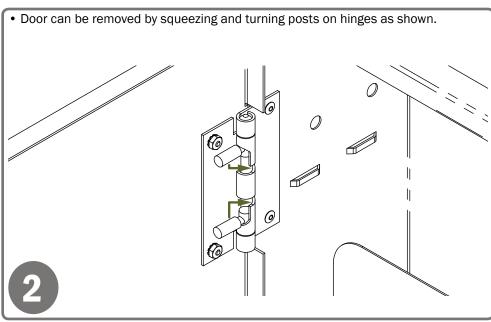

# Parts included:

Dispenser

### **Tools Required:**

· No tools required to assemble dispenser

# Characteristics & Care Powder-Coated Steel

# **Material Characteristics**

# **Designed for Long-Term Use**

Our steel dispensers and stands are coated with polyester powders which makes them highly resistant to corrosion.

Powder-coated steel is ideal for dispensers used in a healthcare or an industrial setting because it is impact resistant and is not effected by extremes in temperature.

Dispensers and stands made of powder-coated steel are both attractive and functional, providing years of service.

# **Care and Cleaning**

For best results, clean by washing the surface with a diluted solution of a mild, non-abrasive detergent in cold or warm water and a non-abrasive cloth. Standard hospital cleaning supplies can be used.

Dry with a soft cloth. Do not use any solvent-based materials, abrasives (e.g. Scotch-brite, steel wool), or apply excessive force during cleaning.