



MYCO
MEDICAL

TheSharpChoice[®]

NMSDC Certified Diversity Supplier

DATASHEET

QLICKSMART[®] BladeNeedleSYSTEM

Blade Remover | Needle Counter



The QCLICKSMART[®] BladeNeedleSYSTEM minimizes the risk of sharps injuries by combining a needle counter with a blade remover to be used in the sterile field.

Performance and Value to Meet Your Safety Needs

The QCLICKSMART[®] BladeNeedleSYSTEM provides a safe way of removing and containing blades and suture needles in operating rooms.

QLICKSMART[®] BladeNeedleSYSTEM Features:

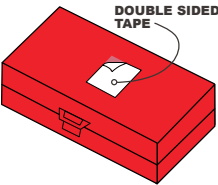
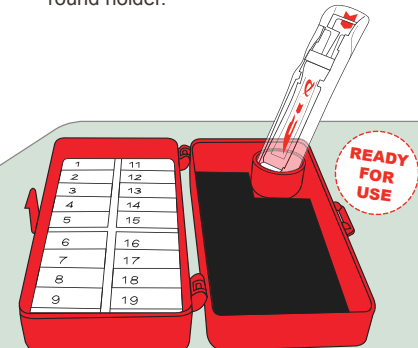
- Needle Counter with an easy-to-use single-handed blade remover
- Remove and contain blades with only one hand
- Safe containment of suture needles
- OSHA Compliant
- Minimize risks of injury and infection
- Compatible with most brands of general surgery blades and flat handles
- Sterile for use in operating rooms



MYCO CODE	DESCRIPTION	UNIT OF MEASURE
QBNS202S-R	BladeNeedleSYSTEM — Sterile	30/box, 4 boxes/case

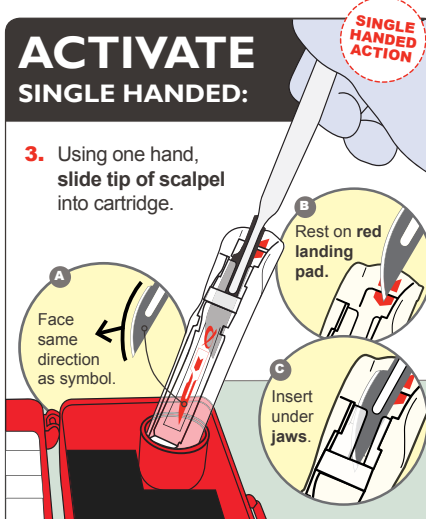
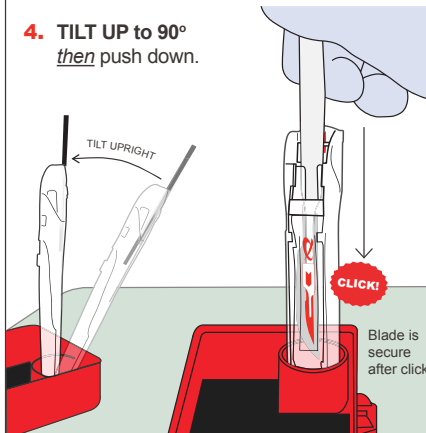
How to Use:

PREPARE USING TWO HANDS:

1. Peel the double sided tapes, open and stick the BladeNeedleSYSTEM down.
 
2. To remove a used scalpel blade, take a cartridge and place it inside the round holder.
 

READY FOR USE

ACTIVATE SINGLE HANDED:

3. Using one hand, slide tip of scalpel into cartridge.
 
4. TILT UP to 90° then push down.
 

SINGLE HANDED ACTION

5. Remove cartridge from the round holder.
 
6. Pull out scalpel handle.
 
7. Ready for counting and later disposal in sharps bin.
 

**SAFE
TO USE TWO HANDS:**

As a leading NMSDC Diversity Certified Supplier, MYCO Medical is committed to “Bringing Reliability, Innovation and Value” to the Healthcare industry with an emphasis on “physician preference” surgical devices and disposables.

To order, learn more or receive complimentary evaluation samples:

Please call MYCO Medical at (800) 454-6926 or (919) 460-2535 or email us at info@mycomedical.com

See the complete line of MYCO innovative disposable and reusable medical devices at www.mycomedical.com



MYCO
MEDICAL
TheSharpChoice®

2015 Production Drive
Apex, NC 27539 USA

info@mycomedical.com
www.mycomedical.com

Some medical devices are not licensed for sale in Canada.

© 2017 MYCO Medical, Inc. All rights reserved.

LIT-00220 Rev 1 Eif 02.17.2017