



MYCO
MEDICAL

TheSharpChoice[®]

NMSDC Certified Diversity Supplier

DATASHEET

QLICKSMART[®] BladeCASSETTE 3-in-1 Blade Remover

The QCLICKSMART[®] BladeCASSETTE is a single-handed, convenient and safe blade removal system.



- OSHA compliant with single-handed activation
- Packaged sterile or non-sterile for flexibility in OR and ER settings
- Compatible with most brands of general surgery blades and flat handles
- Audible click ensures blade removal
- Red arrows show proper insertion location




MYCO CODE	DESCRIPTION	UNIT OF MEASURE
QSSVCAS-3R	BladeCASSETTE — Sterile	100/box, 4 boxes/case
QSSVCAS-BR	BladeCASSETTE — Non-Sterile	500/box

How to Use:

PREPARE


USING TWO HANDS:

- Squeeze the grip ridges on the side of the cassette to release the cover.

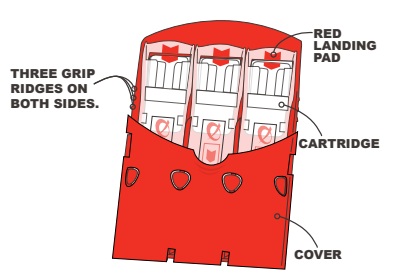

- Fold the cover back and lock in place.

- Place on drape.
 - Nodules help stabilize by gripping on drape.

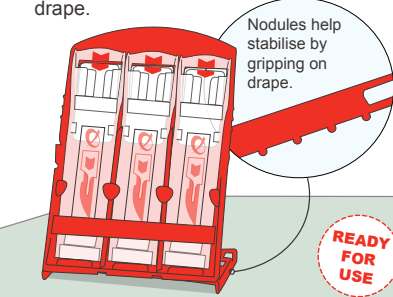


ACTIVATE


SINGLE HANDED:

SINGLE HANDED ACTION

- Using one hand, slide tip of scalpel into cartridge.
 - Rest on red landing pad.

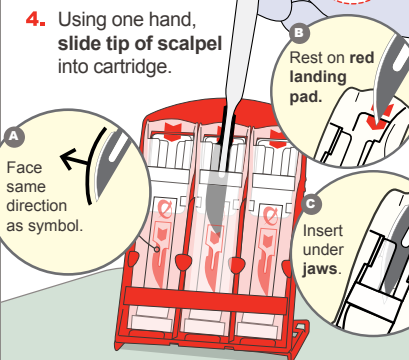


- TILT UP to 90° then push down until you hear a click.

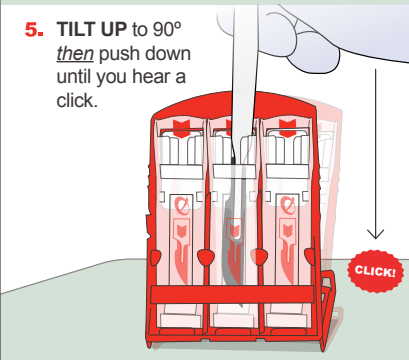


SAFE

TO USE TWO HANDS:

- Brace the BladeCASSETTE with your free hand and pull out scalpel handle.
 - BLADE STAYS SAFE INSIDE CARTRIDGE

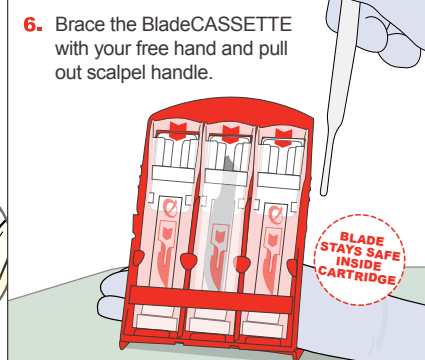

- If necessary, repeat the same procedure to fill the remaining cartridges.
- Ready for counting and later disposal in sharps bin.

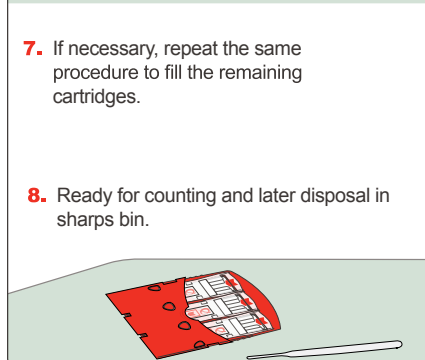

As a leading NMSDC Diversity Certified Supplier, MYCO Medical is committed to “Bringing Reliability, Innovation and Value” to the Healthcare industry with an emphasis on “physician preference” surgical devices and disposables.

To order, learn more or receive complimentary evaluation samples:

Please call MYCO Medical at (800) 454-6926 or (919) 460-2535 or email us at info@mycomedical.com



2015 Production Drive
Apex, NC 27539 USA

info@mycomedical.com
www.mycomedical.com

See the complete line of MYCO innovative disposable and reusable medical devices at www.mycomedical.com

Some medical devices are not licensed for sale in Canada.

© 2017 MYCO Medical, Inc. All rights reserved.
LIT-00218 Rev 2 Eif 07.05.2017