



Sofia®

QUIDEL



QuickStart Guide



Study the User Manual
thoroughly before using
the QuickStart Guide

CUSTOMER SERVICE

If Sofia does not perform as expected, contact Quidel Technical Support 800.874.1517 (in the U.S.), 858.552.1100 (outside the U.S.) or your local distributor. technicalsupport@quidel.com

UNPACK SOFIA

- ▶ Inspect the shipping container for obvious shipping damage prior to opening.
- ▶ Unpack the shipping container and inspect the unit for damage.




Sofia

4x LR6-AA + 6V, === 4.16 A



Paper Roll

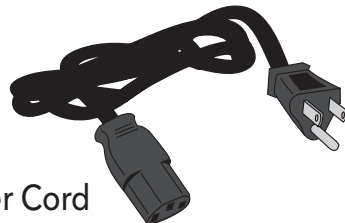

AC Power Cord



DC Power Cord

UNPACK INSTALLATION PACK

- ▶ Inspect the shipping container for obvious shipping damage prior to opening.
- ▶ Unpack the shipping container and inspect the contents for damage.

Barcode Scanner




Barcode Scanner Stand



SD Cards:
Test Type Files (Blue)
Language Files (Yellow)
Blank SD Card



Calibration
Cassette
Pouch

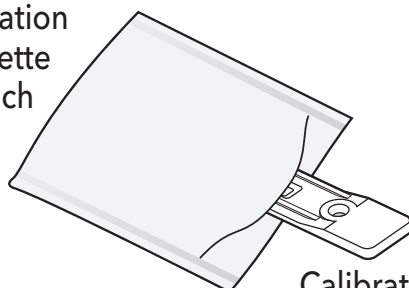


Calibration
Cassette


CONNECT TO POWER AND TURN ON

Place Sofia on the counter top within reach of an electrical outlet. The unit is portable and can be moved to a suitable location for testing.


- 1** Plug the DC power cord into the power port in the back of Sofia.

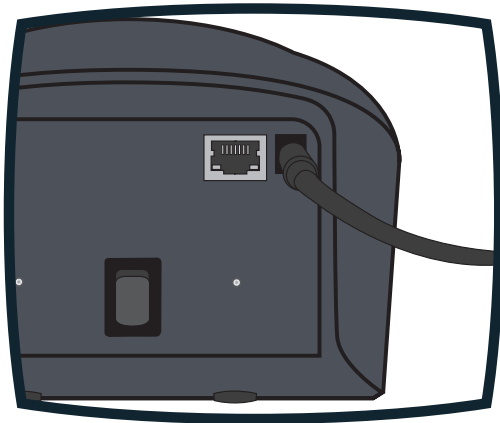


- 2** Plug the appropriate, country specific AC power cord into the DC Power Cord and appropriate electrical outlet.

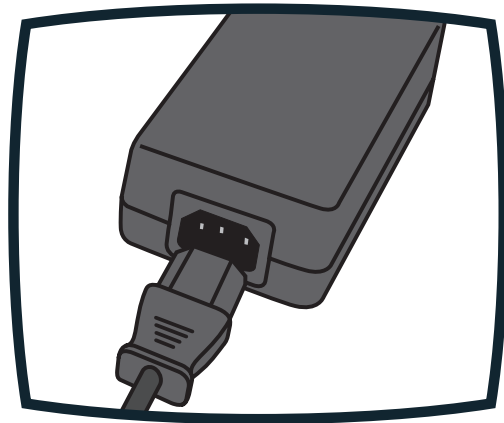

- 3** Plug the Barcode scanner into the PS2 port in the back of Sofia. **Note:** The Barcode scanner may have 2 plugs; one will remain unplugged.

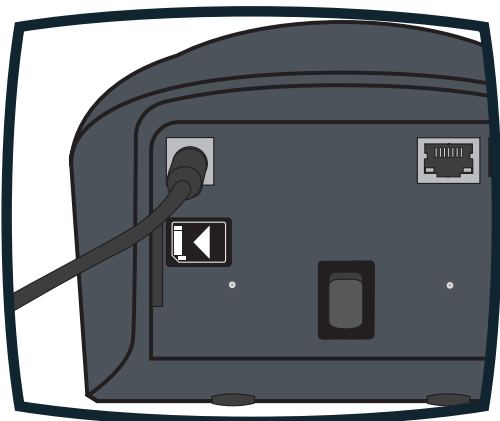

- 4** Turn Sofia on using the power switch located on the rear panel. Hold the toggle switch down until the screen illuminates (1-2 seconds).

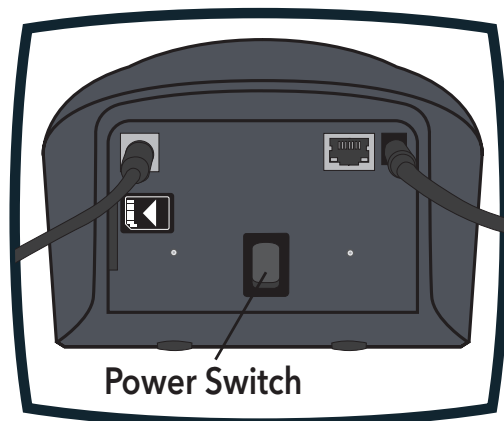


INSTALLING TEST TYPE FILES



Test files must be loaded prior to performing testing. A new test type file is needed for each new assay. Select "Load Test Types" from the Supervisor menu. Then insert the appropriate SD Card with the test type files (blue) into the slot in the back of Sofia. Follow the Sofia prompts to complete the installation.

CALIBRATION CHECK

- ▶ Use the up/down keys to highlight CALIBRATION from the menu.
- ▶ Press the soft key under the lower right corner of the display screen under SELECT.
- ▶ Remove **Calibration Cassette** from the Pouch.

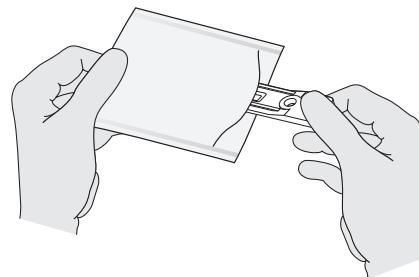

Insert the **Calibration Cassette** into Sofia and close the drawer. Sofia will perform the Calibration Check automatically.

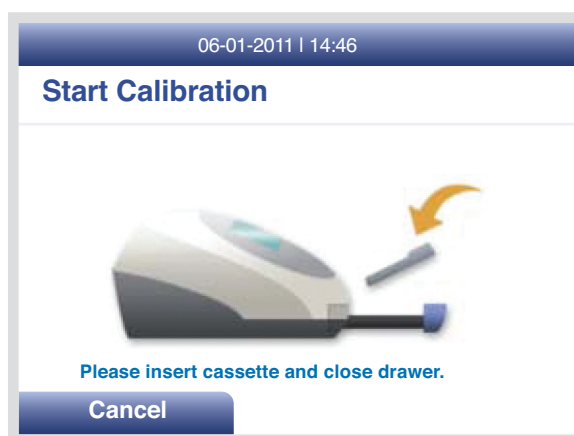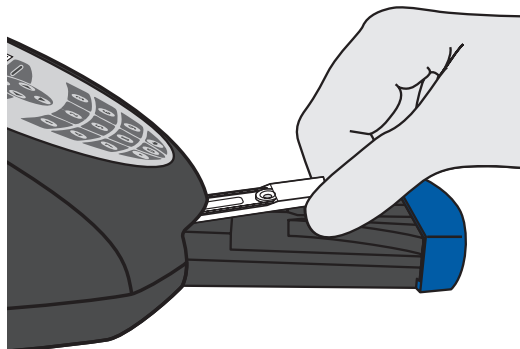

The Sofia display will indicate when the Calibration Check is completed.

NOTE: If Calibration Check cannot be completed successfully, notify the on-site Supervisor or contact Quidel Technical Support for assistance.

The Calibration Cassette will automatically eject when calibration is complete. Remove and return the Calibration Cassette to the storage Pouch.

Select **OK** to return to the Main Menu after calibration is complete.


Important: Store the Calibration Cassette in the provided storage Pouch between uses.

Run QC according to assay specific package insert.



MENU STRUCTURES

Main Menu Structure

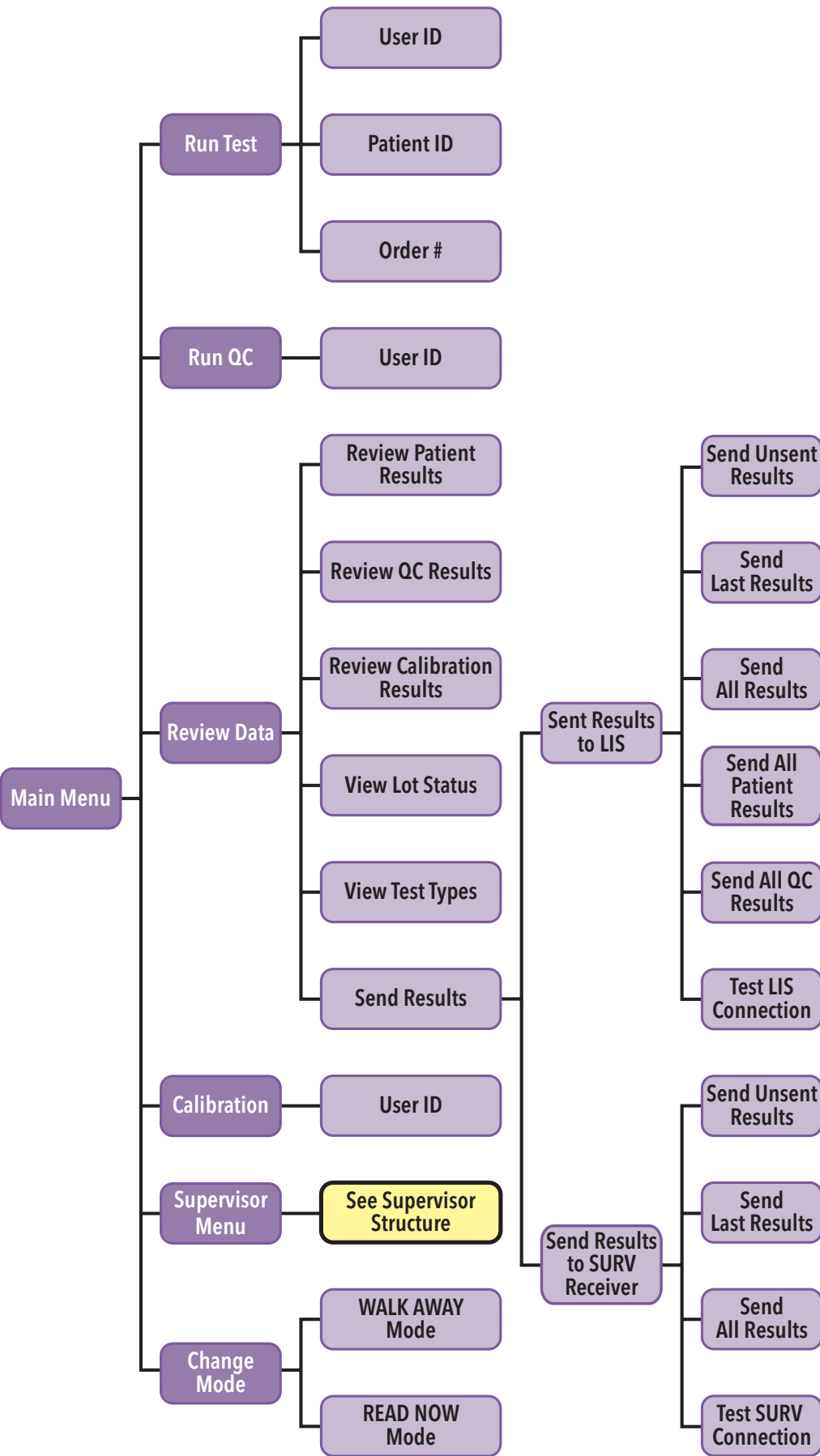

Supervisor Menu Structure

